




Insertion Guide

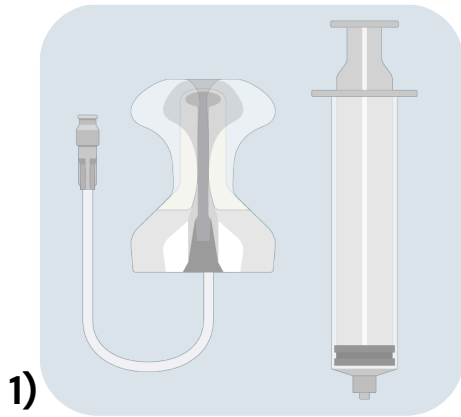


1) Remove the IRD and syringe from packaging and thoroughly lubricate the balloon and exterior surfaces of the IRD handle.

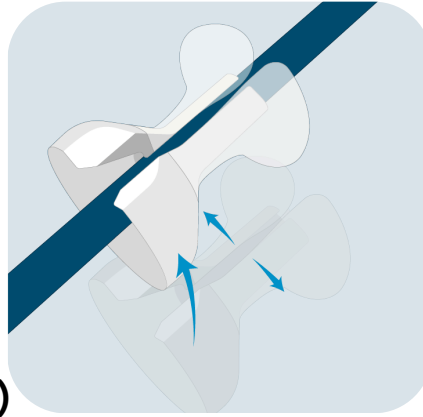


2) Place the IRD fully over the scope by spreading the device apart at the split section as needed.

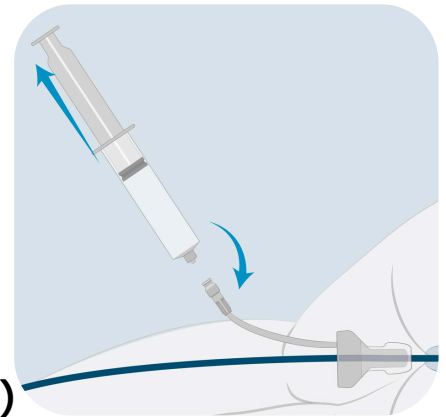


3) Attach syringe to IRD luer valve and retract to remove all air from balloon. Detach syringe from valve.

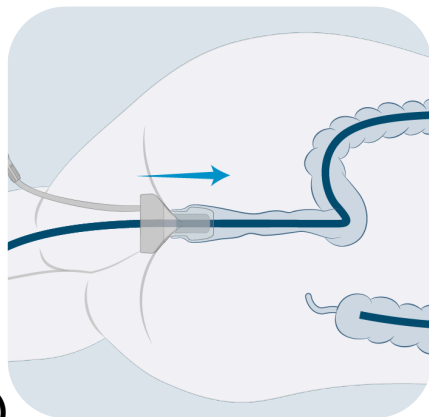

4) Advance the IRD along the scope into the anus. The balloon portion should be within the rectum.

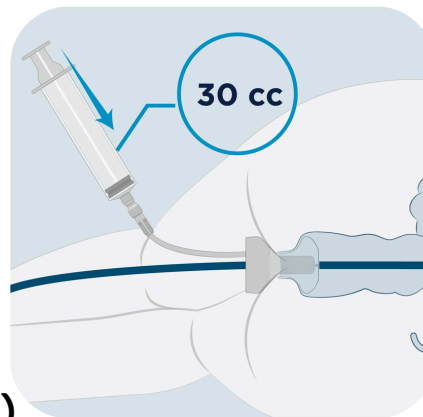


5) Retract syringe plunger rod to 30cc and re-attach syringe to valve. Depress plunger to inflate balloon with 30cc of air. Adjust device positioning and balloon air volume as necessary to achieve an adequate seal with the patient. Do not exceed 50cc total air in the balloon.

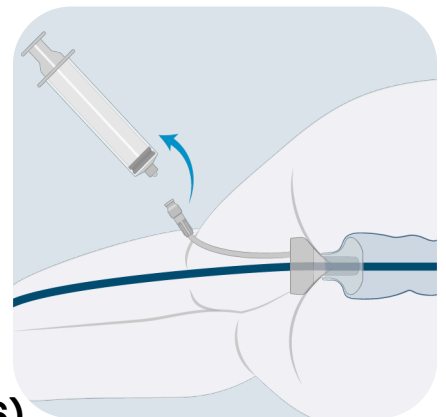


6) Detach syringe from valve.


Removal Guide

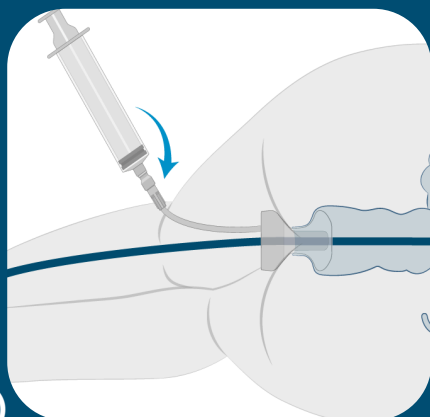

1) Ensure syringe plunger is fully depressed, and reattach syringe to luer valve.

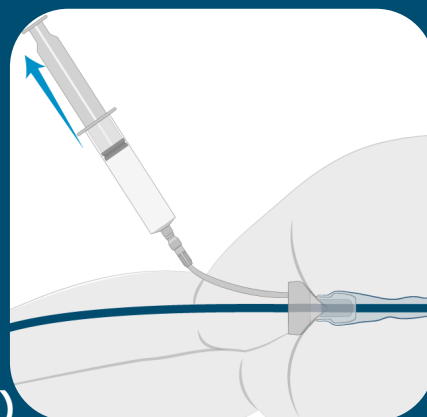

2) Retract syringe plunger, deflating the balloon and loosening the securement of the IRD within the patient.

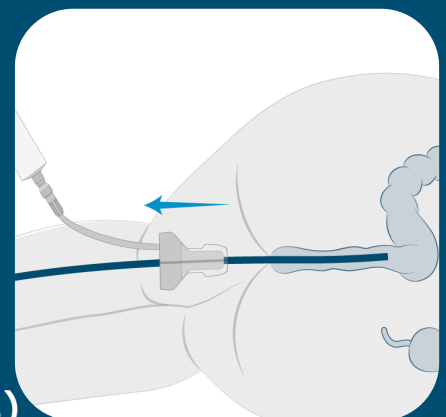

3) Slide IRD along scope to remove from rectum. Remove IRD from scope and dispose with biohazardous waste.